

# Installation of USB driver

Disconnect the IR Transmitter, such as Progress Star, Progress Micro, Control Star, Control Micro. USB/Serial Adapter

Open the file from its current location.

Extract the file and choose a folder to extract to. **Remember the folder.**

Connect the IR Transmitter to the USB port. A message comes up "Found New Hardware" (USB Serial Converter).

An installation Wizard will start.

Mark, Install from a list or from a special location.

Choose the folder that the file has been extracted to. See "Remember the folder" above.

When you are ready a new message comes up, " Found New Hardware" (USB Serial Port).

A new installation Wizard will start.

Mark, Install from a list or from a special location.

Choose the folder that the file has been extracted to. See "Remember the folder" above.

Complete the installation.

## **Gewa AB**

BOX 92, SE-191 22 SOLLENTUNA, SWEDEN

TEL: +46 (0)8-594 694 00 • TEXTTEL: +46 (0)8-594 694 18 • FAX: +46 (0)8-594 694 19

E-MAIL: [info@gewa.se](mailto:info@gewa.se) • WEB: [www.gewa.se](http://www.gewa.se)