

Voice Amplifier

DATA SHEET

062-0130-00 8 SEP 2010

<u>Introduction</u>

The Voice Amplifier uses a 12 watt speaker to provide the loud, clear output required for high quality voice amplification. In addition, it has a LINE input jack that can be connected to the output of a Speech Generating Device (SGD) for adequate sound levels in noisy environments.

The rechargeable battery will provide up to 20 hours of operation from a fully charged battery.




Instructions

- Charge the battery if the LED indicator is dim or off. It requires about 8 hours of charging to bring the battery up to full charge.
 - Turn off the voice amplifier before inserting the power adapter's plug into the Charger Socket.
 - Plug the power adapter into a power source.
 - The LED will strobe red and green while charging. The LED will turn solid red when charging is complete.
 - To extend the battery's life, charge the battery regularly.
- To be worn at the waist, attach the **Voice Amplifier** using the built-in clip or the adjustable waistband provided.

- Put the headband microphone in place and adjust the microphone so it is about 1" (2.5 cm) from the mouth.
 - Insert the microphone's plug into the jack marked MIC. DO NOT PLUG THE MICROPHONE INTO THE LINE JACK!

ATTENTION

Put on the headband microphone before turning on the amplifier.

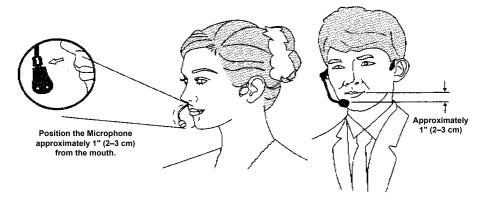

Rotate the Volume control to turn on the Voice Amplifier and adjust the output volume.

ATTENTION: Separation of the microphone and the voice amplifier is necessary to avoid feedback squealing.

- Turn off the **Voice Amplifier** after use.
- A short cable with 3.5 mm plugs on each end is for connecting to sound sources like Speech Generating Devices (SGD)—works well for sound-output entertainment devices, too.

To prevent inadvertent use of the LINE Input jack, it may have a plastic plug installed. Remove the plug to use the LINE Input jack.

Printed in U.S.A

manuals\misc\voice amplifier_1.pmc

ZYGO Industries, Inc

ZYGO USA U.S.A. / Canada: (800) 234-6006 Web site: www.zygo-usa.com

48834 Kato Rd, Suite 101-A, Fremont, CA 94538 TEL: (510) 249-9660 FAX: (510) 770-4930 E-mail: zygo@zygo-usa.com

ZYGO USA U.S.A. / Canada: (800) 234-6006 48834 Kato Rd, Suite 101-A, Fremont, CA 94538 TEL: (510) 249-9660 FAX: (510) 770-4930

Web site: www.zygo-usa.com E-mail: zygo@zygo-usa.com